



ASSEMBLY INSTRUCTIONS

Safer Wheelchair Service Stand



FOR MANUAL WHEELCHAIR USE ONLY


1 General


Check all parts for shipping damage. In case of damage DO NOT USE. Contact Carrier/Safer Automatic Wheelchair Locks for further instructions.


To ensure the safe assembly of the Safer Wheelchair Service Stand, these instructions must be followed.

For Assembly or Technical Support, please call 601-353-3193 x 215

2 Parts Supplied

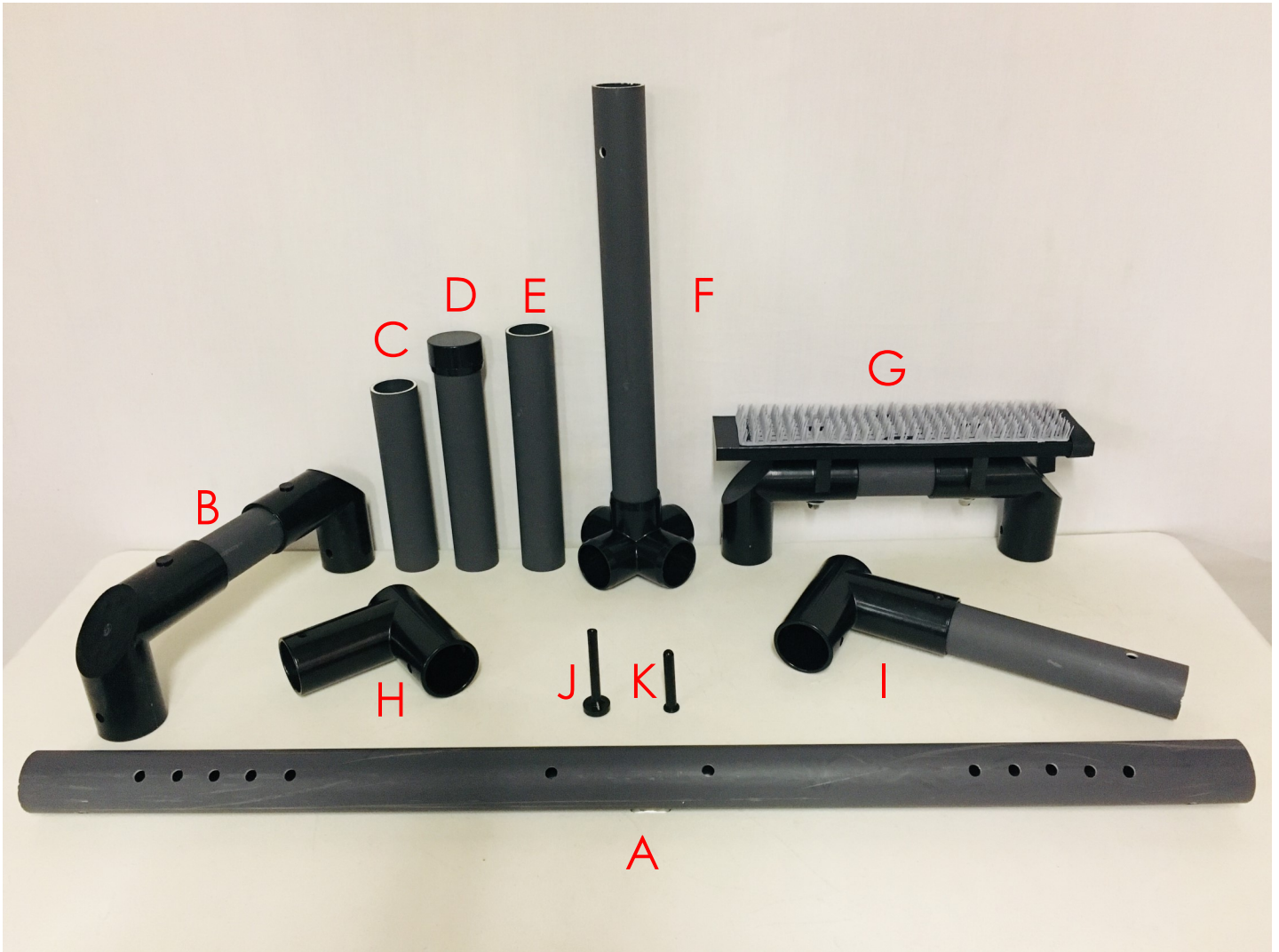


A)	Frame Rails	QTY 2	G)	Chair Supports	QTY 2
B)	Rail Ends	QTY 2	H)	Cut T	QTY 4
C)	Outer Frame Connectors/8.25"	QTY 2	I)	Support Slides	QTY 4
D)	Stabilizers	QTY 8	J)	Pull Pins	QTY 4
E)	Inner Frame Connectors/10.5"	QTY 2	K)	Lock Pins 	QTY 16
F)	Stand Supports	QTY 4	TOOLS REQUIRED: Rubber Hammer		

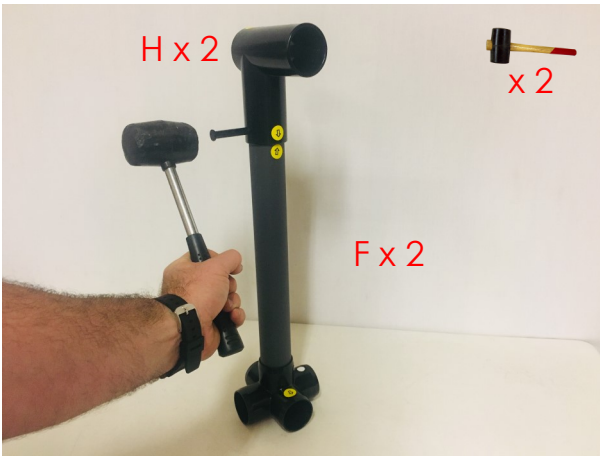

1) Locate the 2 Cut T's (H) and Stand Supports (F) with the Yellow Arrow Labels.

Line up the Yellow Arrow Labels on and connect by using a Rubber Hammer to tap 1 Lock Pin (K) into each.

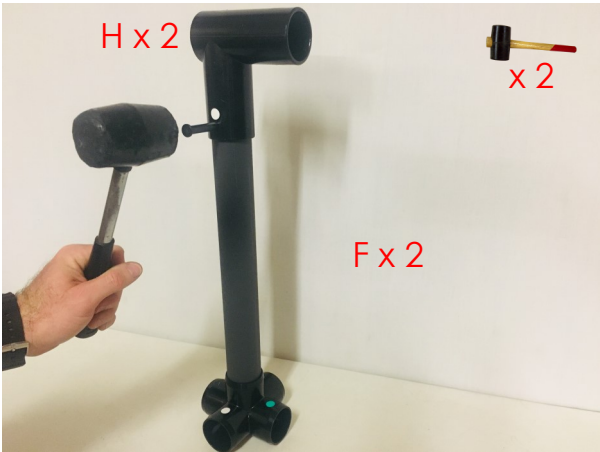


2) Locate the remaining 2 Cut T's and Stand Supports. Line up the White Dots and repeat the process.



3) Attach the two Outer Frame Connectors (C) by color.


(Yellow to Yellow, Green to Green)

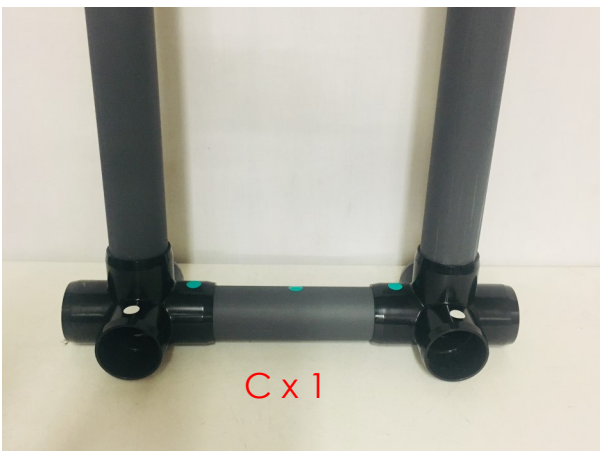




4) Connect the Stand Supports using the 2 Inner Frame Connectors (E).

(White to White)

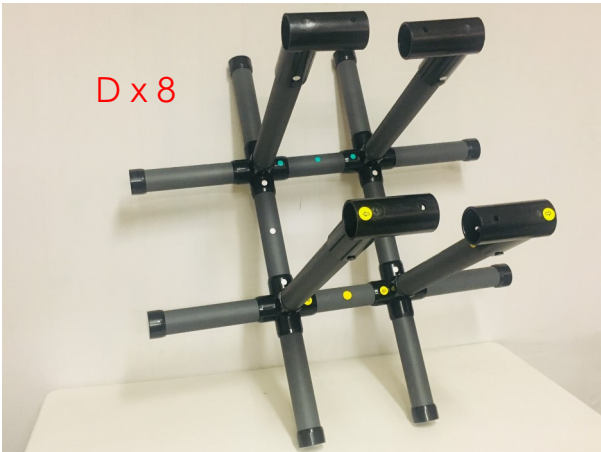


5) Connect the 8 Stabilizers (D) as shown.

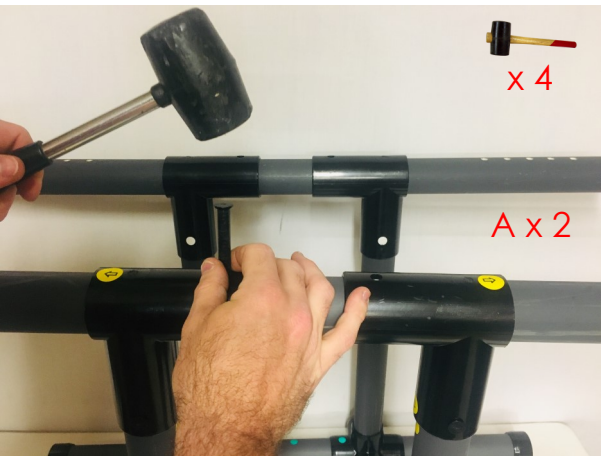

6) Slide the Frame Rails (A) through the Cut T's.

Line up the middle holes on the Rails with the holes at the tops of the Cut T's.

Tap 1 Lock Pin (K) through each hole.
(4 total, 2 on each side)

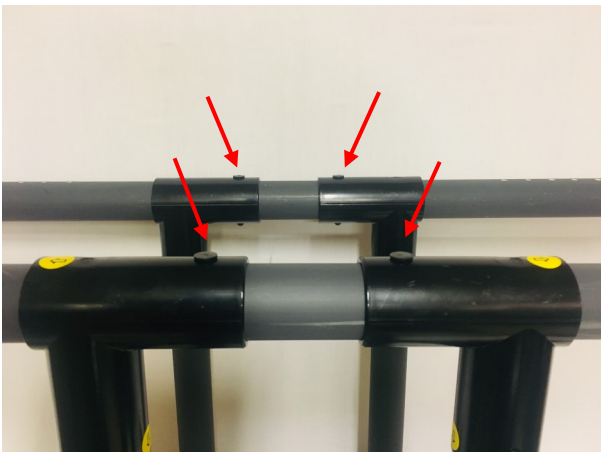




7) Locate the 2 Support Slides (I) with the Yellow Arrow Labels.


Connect 1 of the Support Slides onto each side of the Chair Supports (G) with the Yellow Arrow Label.

Line up the holes and tap 1 Lock Pin (K) through each.

Repeat process for the remaining 2 Chair Supports

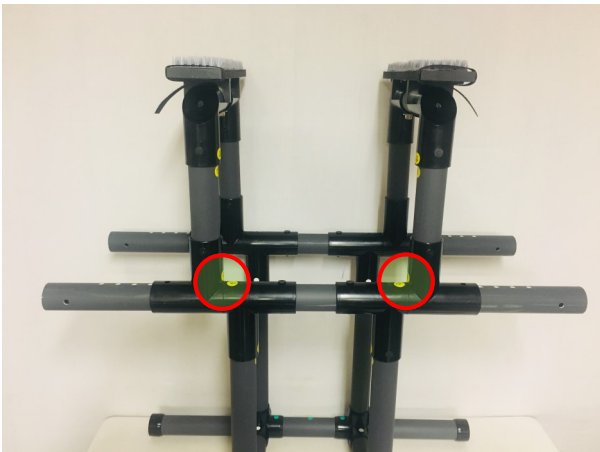

8) Slide the Chair Supports onto the Frame Rails so that the Yellow Arrow Labels are on the same side as shown.

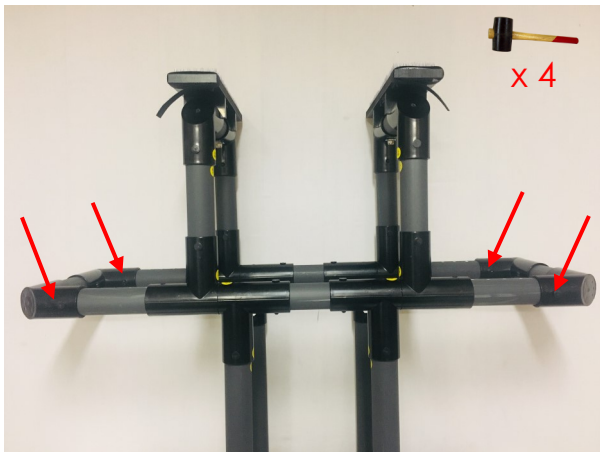

9) Attach Rail Ends (B) to each side.

Tap 1 Lock Pin through each side of both Frame Ends (4 Total)

**** IF USING OPTIONAL PULL HANDLE, ATTACH TO OPPOSITE SIDE OF STAND FROM WHEEL ASSEMBLIES**

SEE NEXT PAGE FOR REFERENCE

IMPORTANT


BEFORE USING:

CHECK THE FOLLOWING & READ INSTRUCTIONS FOR USE ON NEXT PAGE

ALL YELLOW ARROW LABELS MUST BE LOCATED ON ONE SIDE OF YOUR STAND

ALL LOCK PINS ARE TAPPED FLUSH

ALL CONNECTIONS NOT REQUIRING LOCK PINS ARE FULLY INSERTED INTO CONNECTORS

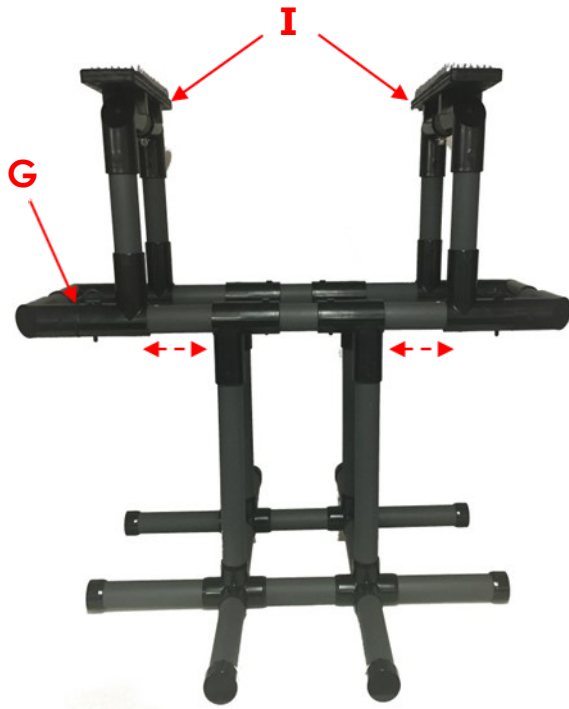

Slide the Chair Supports **equally*** on both Frame Rails to fit the desired width of the wheelchair in use.

***Chair Supports must be spread equally or the stand could become unbalanced and tip over causing personal injury and/or damage to the wheelchair**


Once the Supports are at the correct width, use the Pull Pins (J) to keep the Supports from moving while service is being performed. (4 Pins, 2 for each side)

Use the hook & loop straps (K) to secure the wheelchair.

ACCESSORIES

ACC1 Tool/Parts Bin

ACC2 Transport Package (Includes Caster Assemblies, Pull Handle & Tool/Parts Bin)

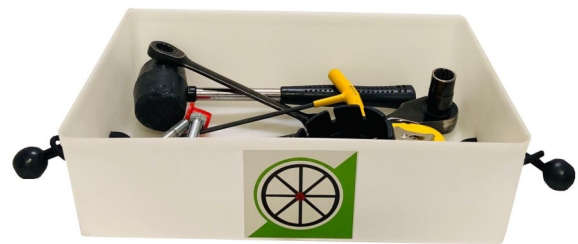

105 E. Rankin Street • Jackson, MS 39201 • 601-353-3193 x 215
www.saferwheelchairs.com